

FIRST – desktop viewer software

Instructions for first use

Version 1 – 20th Nov 2023

This document outlines the personal computer hardware requirements and provides step-by-step instructions for successfully installing and launching the FIRST – Desktop Viewer software on your computer.

System Requirements

- Compatible operating system: Microsoft Windows 10 or newer

Installation and startup

1. Download the compressed folder containing the software files from the RESTECH website (www.restech.it).
2. Create a folder in your "Documents" directory and name it "FIRST – Desktop Viewer."
3. Extract the contents of the compressed folder into the newly created "FIRST – Desktop Viewer" folder.
4. To run the software, double-click the file "resmonfirst-desktop-serial.exe."
5. Follow the instructions provided in Section 11 of the FIRST User Manual to operate the software effectively.

Tips for a seamless usage

- ✓ To simplify the software launch, consider creating a shortcut to the "resmonfirst-desktop-serial.exe" file and placing it on your desktop.
- ✓ Avoid renaming or moving the "FIRST – Desktop Viewer" folder to prevent software malfunction and issues with the shortcut.

Encourage adoption for better respiratory diagnosis

MAKE OSCILLOMETRY ROUTINE